


INSTRUCTIONS FOR OPERATION AND CARE OF

Thermalator™

Moist Heat Therapy Unit

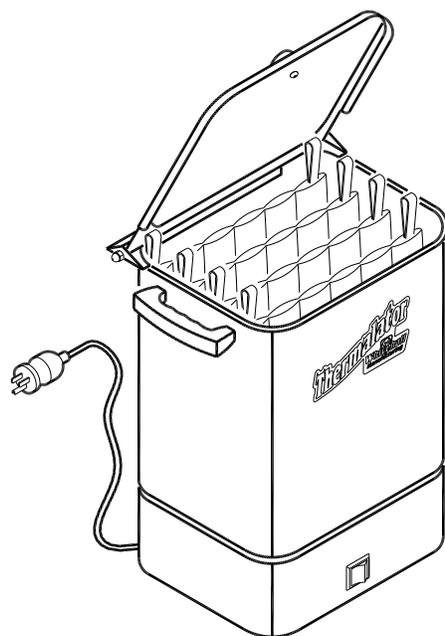

- Stationary 4 Pack
Thermalator
Model T-4-S

- Stationary 4 Pack
Thermalator, 220 Volt
Model ET-4-S



Whitehall Manufacturing®
Manufacturer of Healthcare and Rehabilitation Products since 1946

P.O. Box 3527 • City of Industry, CA, 91744-0527, U.S.A.

800-782-7706 • 626-968-6681 • www.whitehallmfg.com



WARNING



**PLEASE READ THIS ENTIRE
BOOKLET BEFORE OPERATING
YOUR NEW THERMALATOR**



**Failure to follow these instructions could result in damage
to your new heating appliance and/or injury**

European Union CE Mark

The presence of the CE Mark on Whitehall equipment means that it has been designed, tested and certified as complying with all applicable European Union regulations and recommendations.

Waste Electrical and Electronic Equipment (WEEE)

This symbol on the product or on its packaging indicates that this product must not be disposed of with regular waste. Instead, it is the user's responsibility to dispose of waste equipment according to the local laws. Separate collection and recycling of the waste equipment at the time of disposal will help conserve natural resources and ensure it is recycled in a manner that protects human health and the environment. For information about where the user can drop off the waste equipment for recycling, please contact your local waste collection authority. See Page 14 for instructions on how to disassemble the equipment for recycling purposes.

Electric Shock

The Electric Shock Symbol is used to indicate a hazard arising from dangerous voltage. Any mishandling could result in irreparable damage to the equipment and/or personal injury or death.

General Warning or Caution

The Exclamation Symbol appears in Warning and Caution statements. This symbol designates where personal injury or damage to the equipment is possible.



WARNING

To avoid electric shock, connect the instrument to properly earth-grounded, GFCI protected, 3-prong receptacles only. Failure to observe this precaution can result in severe injury.

- DO NOT operate the appliance without properly filling with water. Under no circumstance should the appliance be operated without water, as operating the appliance without water may result in damage to the heating element. Fill appliance with water to the prescribed level before plugging into an electrical receptacle.
- CONNECT the appliance into a properly polarized G.F.C.I. (Ground Fault Circuit Interrupter) electrical receptacle for protection against electrical shock. Have a qualified electrician verify the wall socket is properly polarized and grounded. Use only the power cordset provided with the unit. The Thermalator appliance must be properly grounded.
- AVOID skin contact with HOT water at all times! Carefully place thermal packs into the Thermalator to avoid splashing onto skin or coming into contact with HOT water. DO NOT reach into HOT water with bare hands to remove heated thermal packs. Use a grasping device, such as tongs, to remove thermal packs from HOT water.
- DO NOT use in the presence of flammable liquids or gases, as these may present a fire and/or explosion hazard.
- ALWAYS allow the appliance to cool down before draining water or moving unit.
- DO NOT use the lid knob to lift or transport appliance. Use the side handles to lift or move.
- NEVER leave appliance operating unattended for long periods of time. Water can evaporate and cause appliance to over-heat which may result in damage to the heating element. Turn "OFF" when not in use.



TABLE OF CONTENTS

DESCRIPTION	2
OPERATOR SKILLS AND TRAINING	3
INSPECTING THE THERMALATOR	3
SET UP INSTRUCTIONS	3-6
Setting up	3
Filling my unit	4-5
Emptying my unit	5
USING YOUR THERMALATOR	6-8
How do I use the Thermal Packs?	6-7
How do I change the temperature?	8
How do I use different size Thermal Packs?	8
MAINTENANCE	9
CLEANING	9
STORAGE	9
TROUBLESHOOTING	10
TECHNICAL INFORMATION	11-13
Exploded view	11
Replacement parts (T-4-S)	12
Replacement parts (ET-4-S)	13
Electrical Parts Disassembly	14



Congratulations on receiving your Stationary 4 Pack Thermalator.

Your Thermalator is a simple and trouble-free unit to operate. The tank body, lid, bottom and tank housing are all made from type 304 stainless steel. The controls are an ON/OFF switch on the front of the unit and a thermostat on the rear to control the water temperature. Unit is preset set to a normal operating temperature.


Side handles and a knob on the lid minimize contact with the hot surface. The unit has Thermal Packs with blue and white tabs and a stainless steel rack adapter that fits onto a vinyl coated wire rack. Inside the unit is a heating element and a temperature sensor.

Model T-4-S

100-120 Volts, 50/60 Hz,
8.33 Amps

Model ET-4-S

200-240 Volts, 50/60 Hz,
4.16 Amps

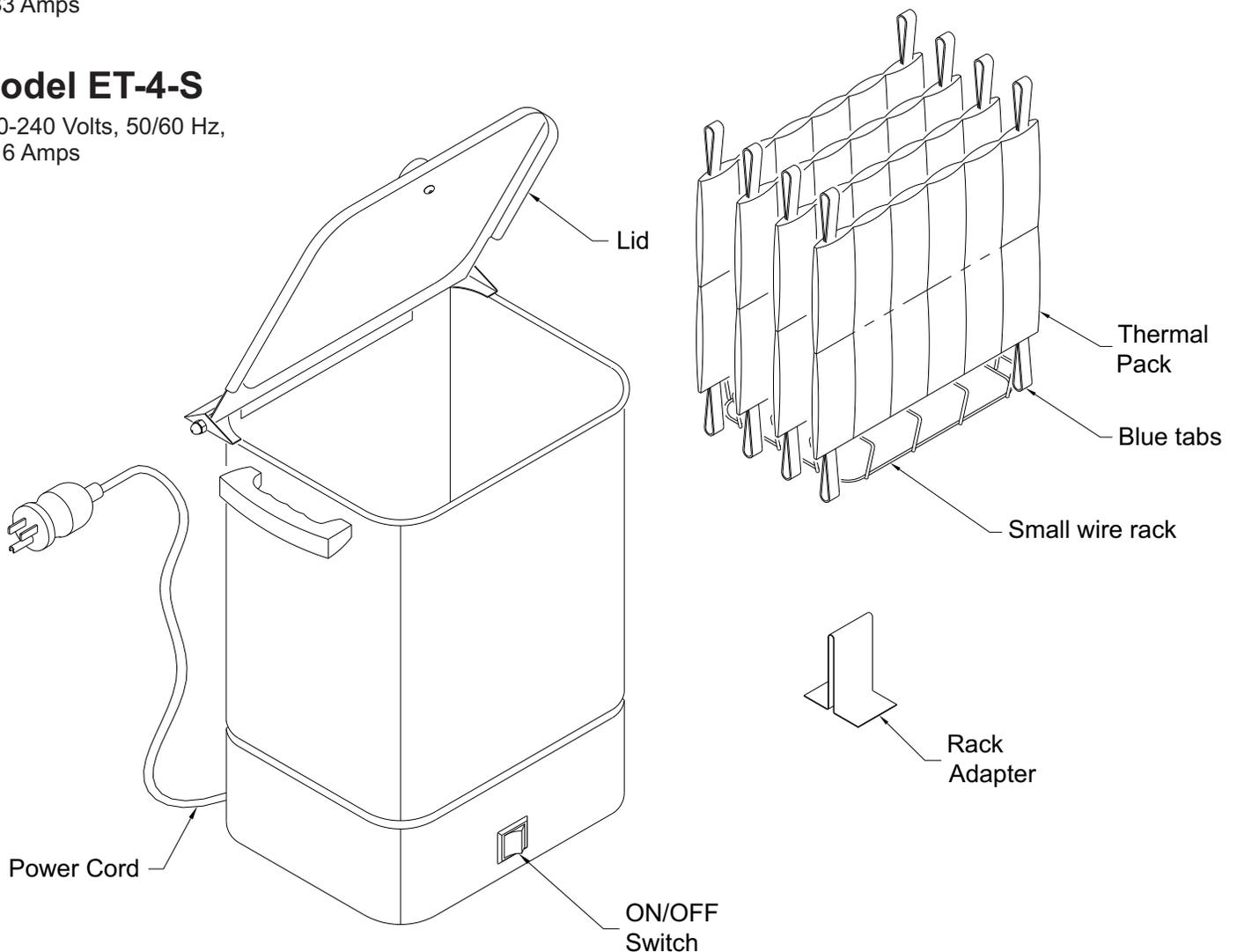



OPERATOR SKILLS AND TRAINING

Skills:

Operators using the Thermalator need a working knowledge of occupational therapy procedures.

WARNING

Only trained personnel to operate the Thermalator. Untrained operators can cause injury or be injured.

TRAINING

Operator trainees need to:

- be trained in occupational therapy protocols.
- read and understand this manual.

INSPECTING THE THERMALATOR

This Whitehall product has been carefully packaged at the factory to minimize the possibility of damage during shipping.

- Inspect the packaging for external signs of damage.
- Inspect the contents for damage.

If there is visible damage to the instrument upon receipt, inform the shipping company and Whitehall immediately.

WARNING

Do not attempt to operate this equipment if there is evidence of shipping damage or you suspect the unit is damaged. Damaged equipment may present additional hazards to you. Contact Whitehall technical support for advice before attempting to plug in and operate damaged equipment.

Inspection Checklist

- Is the Thermalator free of excessive wear?
- Is a properly-grounded and voltage-matched receptacle available?
- Is the outlet equipped with a functioning GFI?

SETTING UP

1. Your new Thermalator includes the following:

- 1 Thermalator Model T-4-S or ET-4-S
- 4 Thermal Packs (4 Standard)
- 1 Wire Rack - Small, Vinyl Coated
- 1 Rack Adapter (stainless steel)
- 1 Thermostat Knob
- 1 Warranty Registration Card
- 1 Instructions for Operation and Care

If you are missing anything, please call the phone number listed on the back page.

2. Unpacking your Thermalator

Remove all the shipping material. Remove Thermal Packs and wire rack from inside the tank. Remove all packaging from inside the tank, especially from under the heating element inside the tank. Clean the unit thoroughly before using.

3. Device Placement

Place the Thermalator at a convenient level, such as on a table or a scratch resistant counter top. Before moving the Thermalator, unplug and cool the unit. Only move the unit to empty or fill it. Always use the handles. The unit may be hot to the touch.


4. Pre-Soaking Thermal Packs

Before the first use, you must condition each Thermal Pack.

- 1- Take the Thermal Pack by the corner loops - one white and one blue or black - so the sections are horizontal.
- 2- Gently shake the pack back and forth to evenly distribute the filler material.



Shake to distribute the dry material

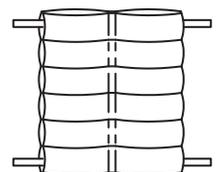




3- Place the Thermal Packs in a large pan to pre soak (use plastic, stainless steel or enamel-ware, DO NOT use aluminum). You may use the Thermalator for pre soaking.

- Place the Thermal Pack in the pan or unit with sections horizontal.

4- Fill the container with enough tap water to completely cover the Thermal Packs.

5- Soak the packs for 24 hours minimum, more is recommended.


6- Return the packs to upright position. (If the unit was used, change the water.) The packs are now ready for heating.

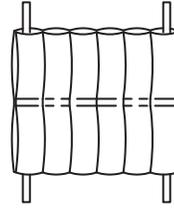


Turn the packs to upright as shown

FILLING your unit


- 1 Make sure the unit is unplugged and all packaging has been removed.

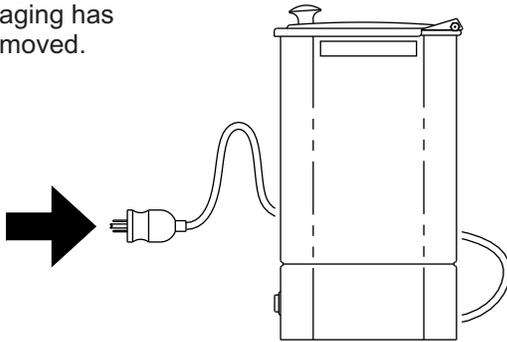


- 2 Fill unit about 3/4 full with tap water.

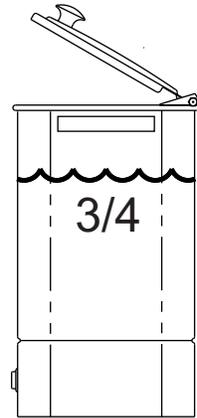

- 3 Place the wire rack inside the unit. Be careful of the heating element.

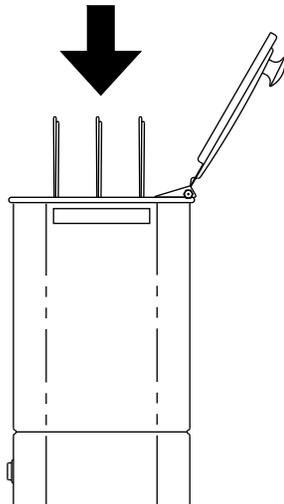


- 4 Put the Pre-Soaked Thermal Packs on the wire rack. Place all the blue or white tabs up.

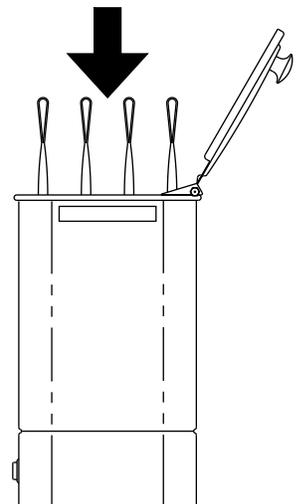




- 5 Add more water, if necessary, to cover the Thermal Packs. Uncovered packs may be damaged during heating.

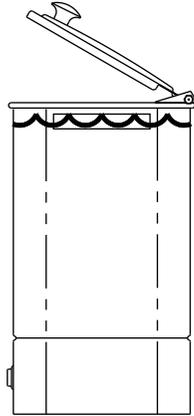


- 6 Close the lid.




Caution! Lid can pinch fingers.

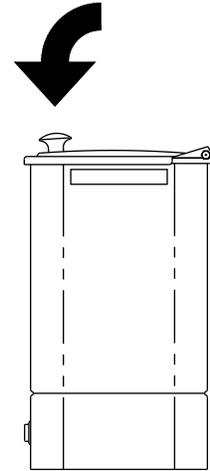

- 7 Plug the power cord into an AC outlet. A Ground Fault Circuit Interrupter is recommended because it will give additional protection from electric shock.

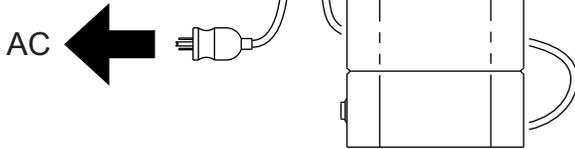


- 8 Facing the front of the unit, turn the power switch to the ON position. Indicator light should be illuminated.

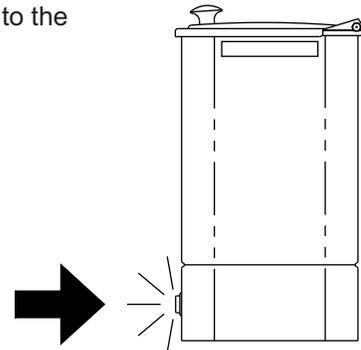


EMPTYING your unit

- 1 Facing the front of the unit, turn the power switch OFF. The light will go out.

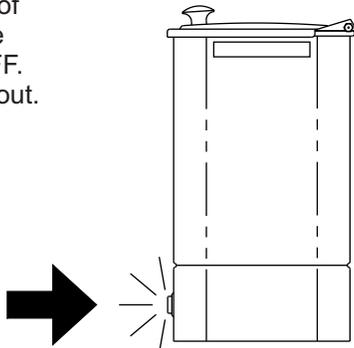


- 2 Unplug the unit.

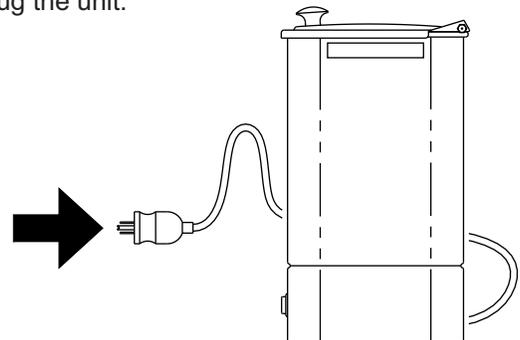




- 3** Let the water cool to room temperature. Remove the Thermal Packs and the wire rack.

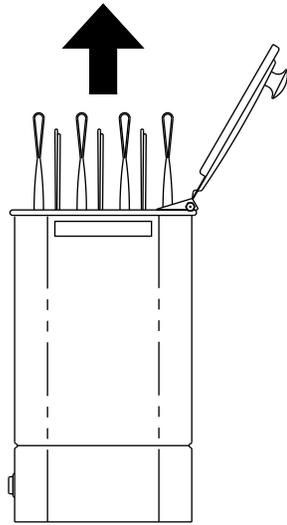


- 4** Pick up the unit by the handles and pour out into a sink or floor drain.

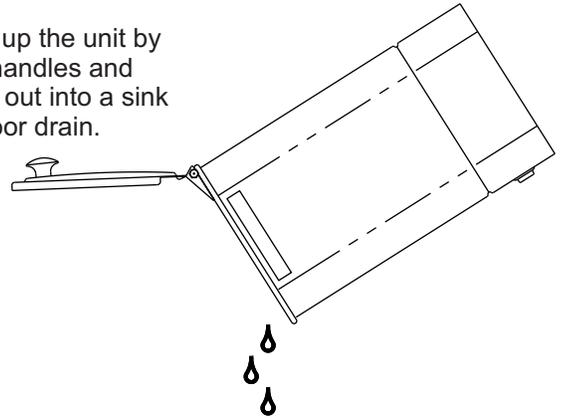

USING the Thermal Pack


1- For each use:

- 1** Open the lid.

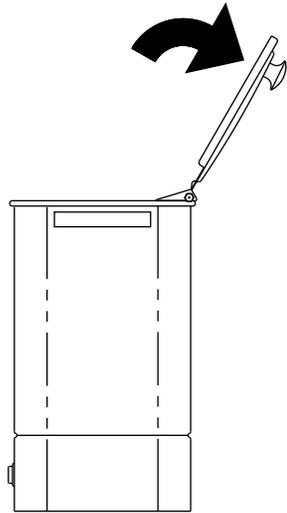


- 2** Using tongs, carefully remove the Thermal Pack.



Caution: the Thermal Pack can cause skin burns.

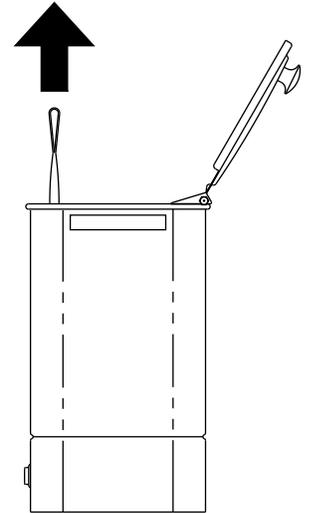

- 3** Wrap the pack in a Thermal Pack Cover or 5 to 6 layers of terry toweling.

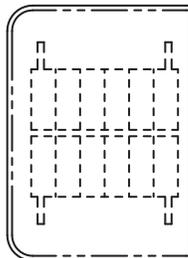

- 4** Put the wrapped pack on the area to be treated for a pre-set treatment time.

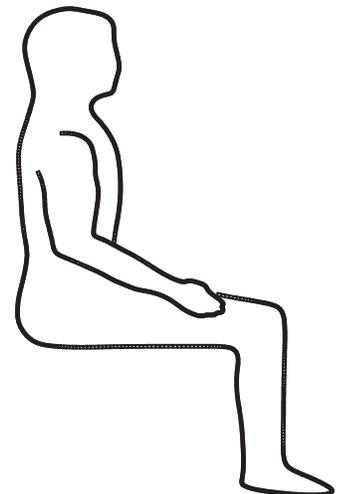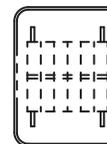


WARNING


Use the Thermalator only as directed in this manual. Any other use can cause injury.



5 Remove the Thermal Pack from the patient.

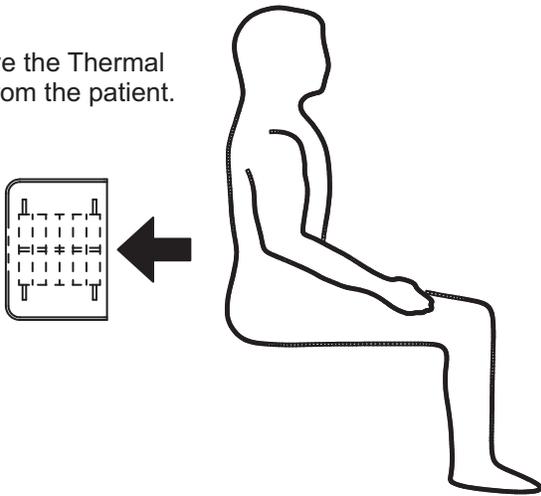


6 Remove the Thermal Pack Cover or toweling. Hang to dry or launder.

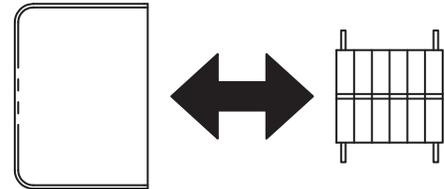


7 Return the pack to the Thermalator.

If the blue tab was up, put the white tab up now. Work your way from first to last, reversing tabs each time.

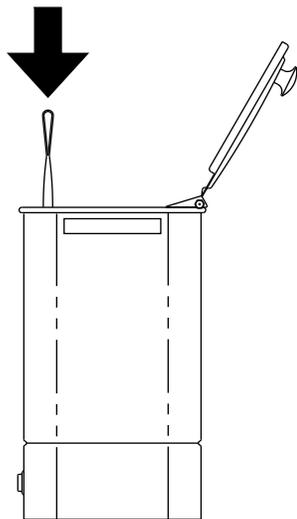


8 Close the lid. The Thermal Pack will be reheated within 15 minutes and ready for use.

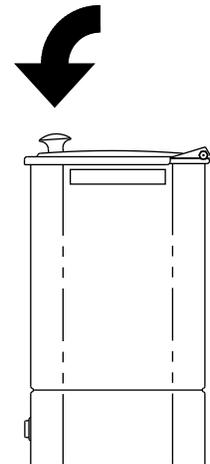


2- Daily use:

1 Add more water, if necessary, to cover the Thermal Packs. Uncovered packs may be damaged during heating.

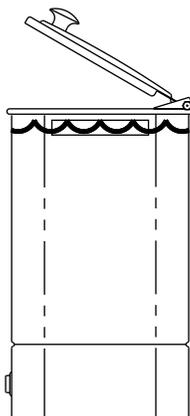

2 Place all the blue tabs in the same reference (either all up or all down) to start the day.

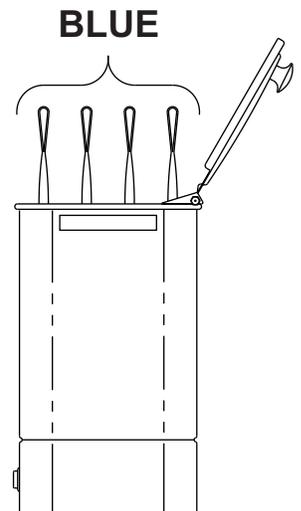




3- Periodic use:


The water temperature for normal operating conditions is about 165°F (74°C). The thermostat is very sensitive; even a small adjustment can raise or lower the temperature several degrees. The thermostat has an adjustable temperature range of 65°F (18°C) to 175°F (79°C) ±5°F (-15°C). To adjust the temperature, you need a Thermometer capable of 180° F (83° C).

- 5 Facing the back of the unit, find the metal temperature stem between the 2 screws in the lower middle of the unit. Attach knob.

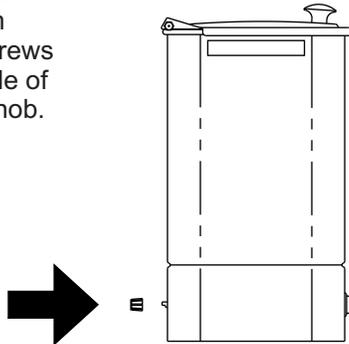


- 6 Using the knob, turn clockwise to increase the temperature. Allow the temperature to stabilize, and measure the water temperature with the thermometer.

CAUTION: Sensitive thermostat - a small adjustment changes several degrees. Temperature range of 65°F (18°C) to 175°F (79°C) ±5°F (-15°C).

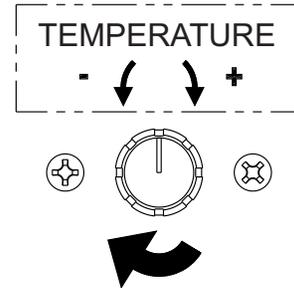

USING the Wire Rack

Your Thermalator uses many different sizes of Thermal Packs. For Half, Cervical or Myofacial sizes, use the adapter as shown. For Standard, Knee/Shoulder, 10" x 18" and 10" x 24" sizes, use only the wire rack.

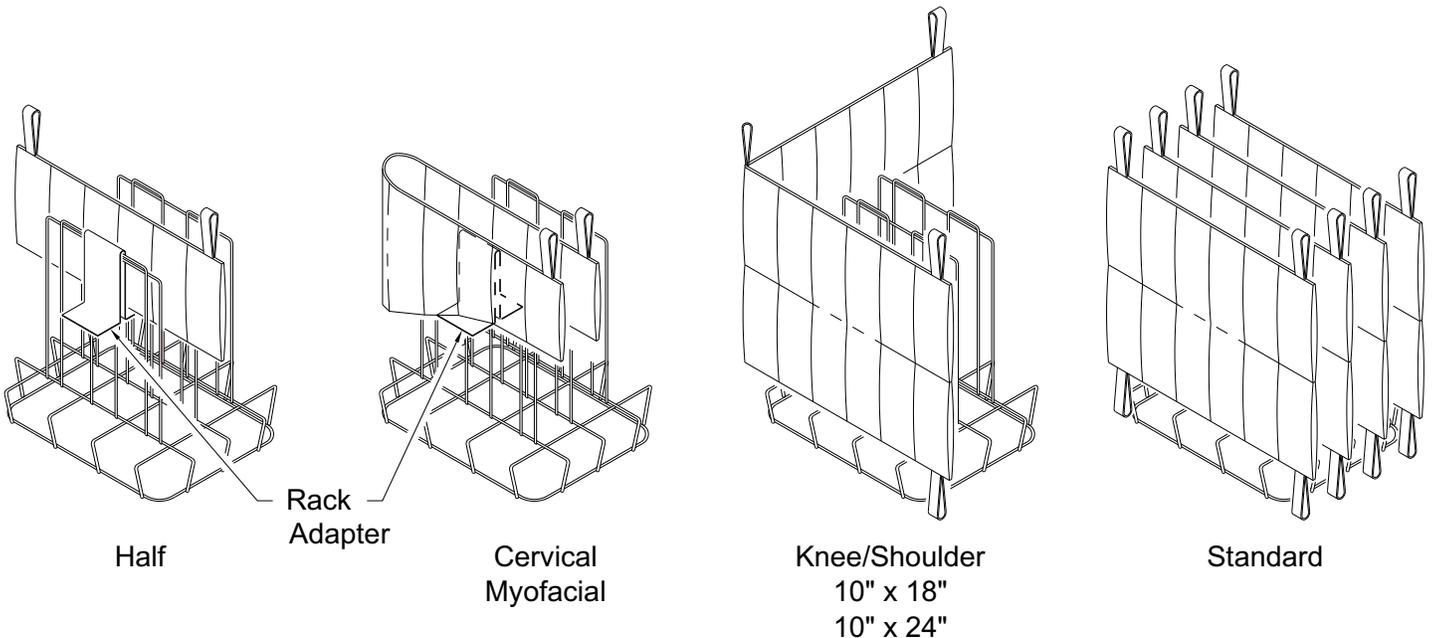



MAINTENANCE

The Thermalator design is nearly maintenance-free. It should, under most conditions, give many years of reliable service. Clean and replace the Thermal Packs as described below.

1. Routine Maintenance (As Needed)

- See Care and Cleaning
- Always keep the Thermal Packs under water. Replace any water that has evaporated.
- Destroy or discard any contaminated Thermal Packs.

2. Monthly Maintenance

- See Care and Cleaning
- Clean wire rack and insert using mild soap. Rinse thoroughly with clean tap water. Dry with soft cloth.
- Scrub your Thermal Packs with warm soap and water.
- Keep the pH (a measure of acidity of the water) between 7.4 and 7.8 to reduce the scaling and residue build-up.

3. Yearly Maintenance

- Replace worn out Thermal Packs. When The packs begin to wear out, the non-toxic filler oozes through the fabric and seams. The individual cells seem empty. These packs don't hold the heat. Throw away worn Thermal Packs.
- Have a professional electrician thoroughly check the following:
 - 1- Test for leakage current, dielectric strength, and receptacle polarity and ground. If you need test details, call the number listed on the back page.
 - 2- Check the power cord and plug. They should be free of cuts, abrasion and other damage. Replace the power cord assembly if it is damaged. Use only factory authorized replacement parts.
 - 3- Remove the bottom cover and check all electrical components and connections. Replace any parts that are in questionable condition. Use only factory authorized replacement parts.

CARE AND CLEANING

Before attempting to clean the appliance, be sure to turn power switch "OFF" and unplug the unit from the electrical receptacle. If necessary, empty (drain) the appliance by following instructions as outlined in the "EMPTYING your unit" section.

Use a soft cloth, sponge, and/or non-abrasive cleaning pad to clean & wipe down the appliance. DO NOT use steel wool to clean the stainless steel surfaces since it can cause rusting on the stainless steel surfaces.

Use mild soaps, gentle non-abrasive detergents, or mild mixtures of ammonia to routinely clean the surfaces of the appliance. Be sure to always rinse with clean, warm water and wipe dry.

Bleaches or iodine should not be used, if they are, rinse thoroughly. Never allow bleach or iodine to "soak" on the stainless steel surface. Continued use will result in pin holes in the stainless steel tank and cause leaking.

STORAGE

1. Storing the Thermalator

- See Care and Cleaning
- Place the Thermalator upright in a dry area away from any chemicals.

2. Storing the Thermal Packs

- Scrub your Thermal Packs with warm soap and water.
- Moist and Cold
 - 1 - Put the moist Thermal Pack in a plastic bag.
 - 2 - Seal the bag.
 - 3 - Store the bag in the freezer (recommended) or refrigerator.
- Dry
 - 1 - Air dry the Thermal Pack completely in a well ventilated area. The filler will become hard and caked. This doesn't hurt the bags; more Pre-Soaking time is then necessary.
 - 2 - Store in a dry area away from any chemicals.



TROUBLESHOOTING


Problem	Cause	Solution
Water doesn't get hot, even though unit is plugged in and switch is ON	Circuit breaker is off Thermostat is not working Heating element is burned out	Reset circuit breaker Replace the thermostat Replace the element
Water is cloudy	The Thermal Packs are worn out (seeping) Too long between cleaning	Replace the Thermal Packs Drain, clean and refill
Thermal Packs too hot	The thermostat is set too high The thermostat is broken	Adjust the thermostat Replace the thermostat
Thermal Packs too hot and then too cool	The thermostat is broken	Replace the thermostat
Thermal Packs too cool	The thermostat is set too low The thermostat is broken There is a power failure	Adjust the thermostat Replace the thermostat Check the circuit and turn on the breaker
Rust in unit	Contact with ferrous metal	Remove the ferrous metal item from the unit- usually a staple, paper clip, safety pin, etc. Clean thoroughly.

REPAIR OF UNIT

It may be necessary or you may desire to return the Thermalator unit for service. In that case, we would appreciate you contacting Whitehall Manufacturing directly. We will make arrangements for the unit to be returned to our production location. When the unit is received by us, an estimate of repairs will be made and you will be contacted. **Please- we appreciate warranty repair or any other repair be arranged directly with Whitehall Manufacturing. If you are not satisfied with our service, then and only then, contact your dealer.**




Exploded View

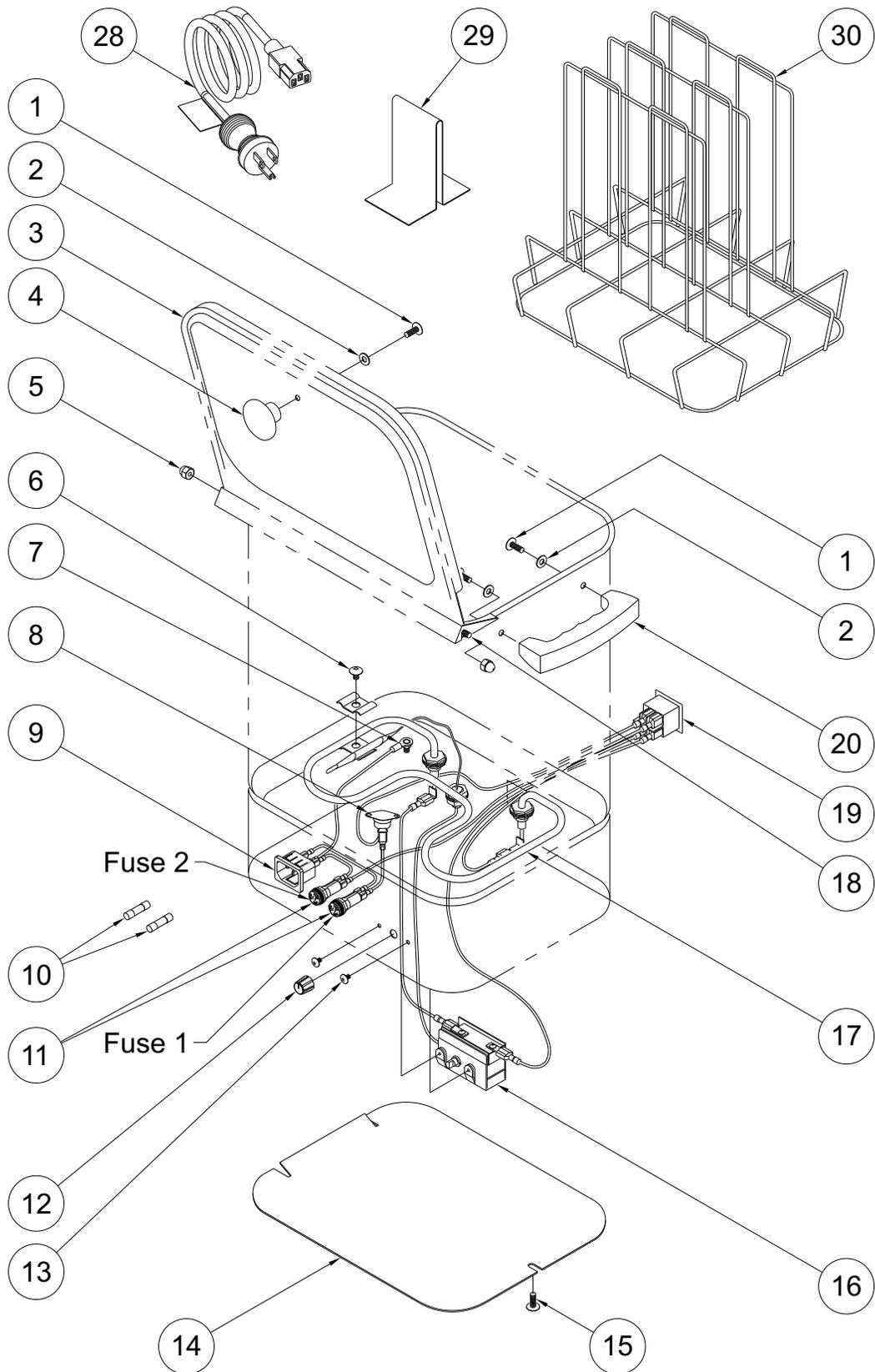



Model T-4-S (120 VAC 60 Hz)

ITEM	PART NUMBER	DESCRIPTION
1	6502-202-000	#10-32 x 1/4" phillips screw
2	0411-021-000	Fiber washer
3	6702-030-199	Lid
4	6702-060-000	Knob
5	6502-524-000	#10-32 acorn nut
6	6702-050-002	Thermometer Clamp Assembly
7	0302-010-000	#6-32 hex nut
8	0710-446-000	Thermal Overheating Sensor
9	6505-546-000	10A/15A 250VA Snap-in power inlet
10	6900-136-000	Fuse 15 A, Ceramic, Time Delay, 1/4 x 1-1/4" Lg
11	0710-232-000	Fuse holder assembly 1/4 x 1-1/4" Lg
12	6505-508-000	Thermostat knob
13	0116-008-000	#6-32 x 3/16" phillips screw
14	6702-025-199	Bottom cover
15	0116-011-000	#10-32 x 5/16" phillips screw
16	6505-511-000	Thermostat
17	6505-500-000	Heating Element (120VAC/1000W) w/ washers & nuts
18	6503-503-199	Hinge pin
19	6505-510-000	On/Off switch (DPST w/ lamp)
20	6702-061-000	Handle - small
21	6704-092-001	Wire connector assembly, black 6"
22	6704-091-001	Wire connector assembly, black 5"
23	6704-090-001	Wire connector assembly, black 9"
24	6704-098-001	Wire connector assembly, white 4"
25	6704-086-001	Wire connector assembly, green/yellow 6"
26	6704-099-001	Wire connector assembly, white 9"
27	6704-100-001	Wire connector assembly, white 6"
28	6505-056-000	Power cord 13A, 125V
29	6702-045-199	Rack adapter (stainless steel)
30	6702-040-199	Wire rack - small

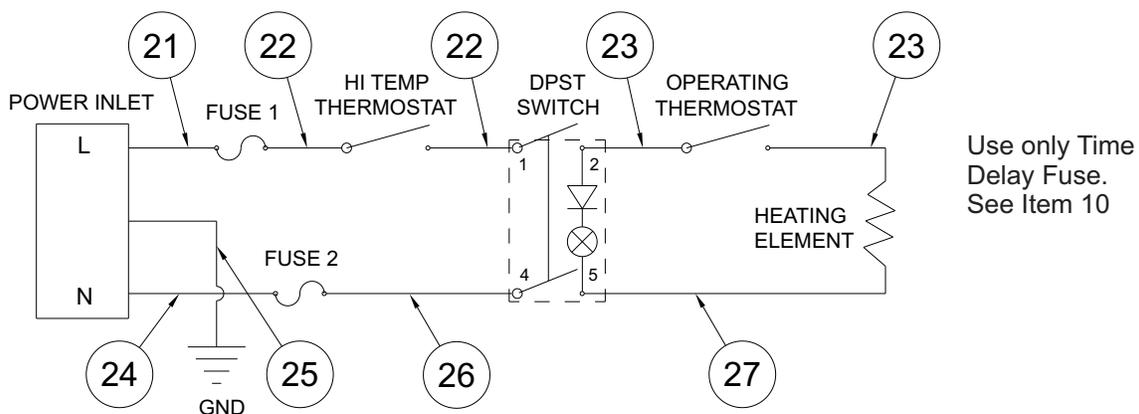

Protective Conductor Terminal

The protective conductor terminal symbol in the above figure identifies the location of the bonding terminal, which is bonded to conductive accessible parts of the enclosure for safety purposes.


Symbols on labels:

- Alternating Current
- Fuse Location



Model ET-4-S (220 VAC 50/60 Hz)

ITEM	PART NUMBER	DESCRIPTION
1	6502-202-000	#10-32 x 1/4" phillips screw
2	0411-021-000	Fiber washer
3	6702-030-199	Lid
4	6702-060-000	Knob
5	6502-524-000	#10-32 acorn nut
6	6702-050-002	Thermometer Clamp Assembly
7	0302-010-000	#6-32 hex nut
8	0710-446-000	Thermal Overheating Sensor
9	6505-546-000	10A/15A 250VA Snap-in power inlet
10	0710-202-000	Fuse 6.3 A, Ceramic, Time Delay, 5mm x 20mm
11	0710-230-001	Fuse holder assembly 5mm x 20mm
12	6505-508-000	Thermostat knob
13	0116-008-000	#6-32 x 3/16" phillips screw
14	6702-025-199	Bottom cover
15	0116-011-000	#10-32 x 5/16" phillips screw
16	6505-511-000	Thermostat
17	6505-501-000	Heating Element (220VAC/1000W) w/ washers & nuts
18	6503-503-199	Hinge pin
19	6505-516-000	On/Off switch (DPST w/ lamp)
20	6702-061-000	Handle - small
21	6704-073-001	Wire connector assembly, brown 5"
22	6704-074-001	Wire connector assembly, brown 5"
23	6704-072-001	Wire connector assembly, brown 9"
24	6704-083-001	Wire connector assembly, blue 4"
25	6704-086-001	Wire connector assembly, green/yellow 6"
26	6704-071-001	Wire connector assembly, blue 9"
27	6704-070-001	Wire connector assembly, blue 6"
28	Call for part #	Power cord
29	6702-045-199	Rack adapter - Cervical (stainless steel)
30	6702-040-199	Wire rack - small

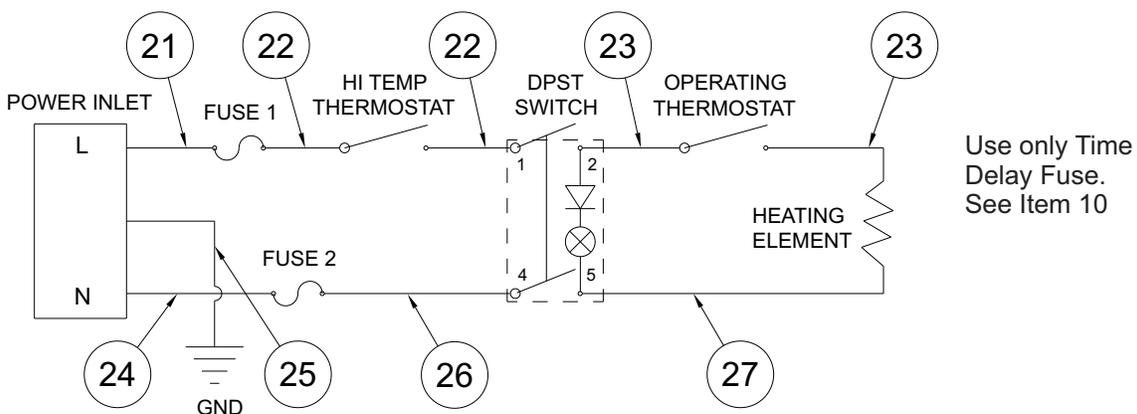

Use only Time Delay Fuse. See Item 10



Protective Conductor Terminal


The protective conductor terminal symbol in the above figure identifies the location of the bonding terminal, which is bonded to conductive accessible parts of the enclosure for safety purposes.

Symbols on labels:

- Alternating Current
- Fuse Location



ELECTRICAL PARTS DISASSEMBLY

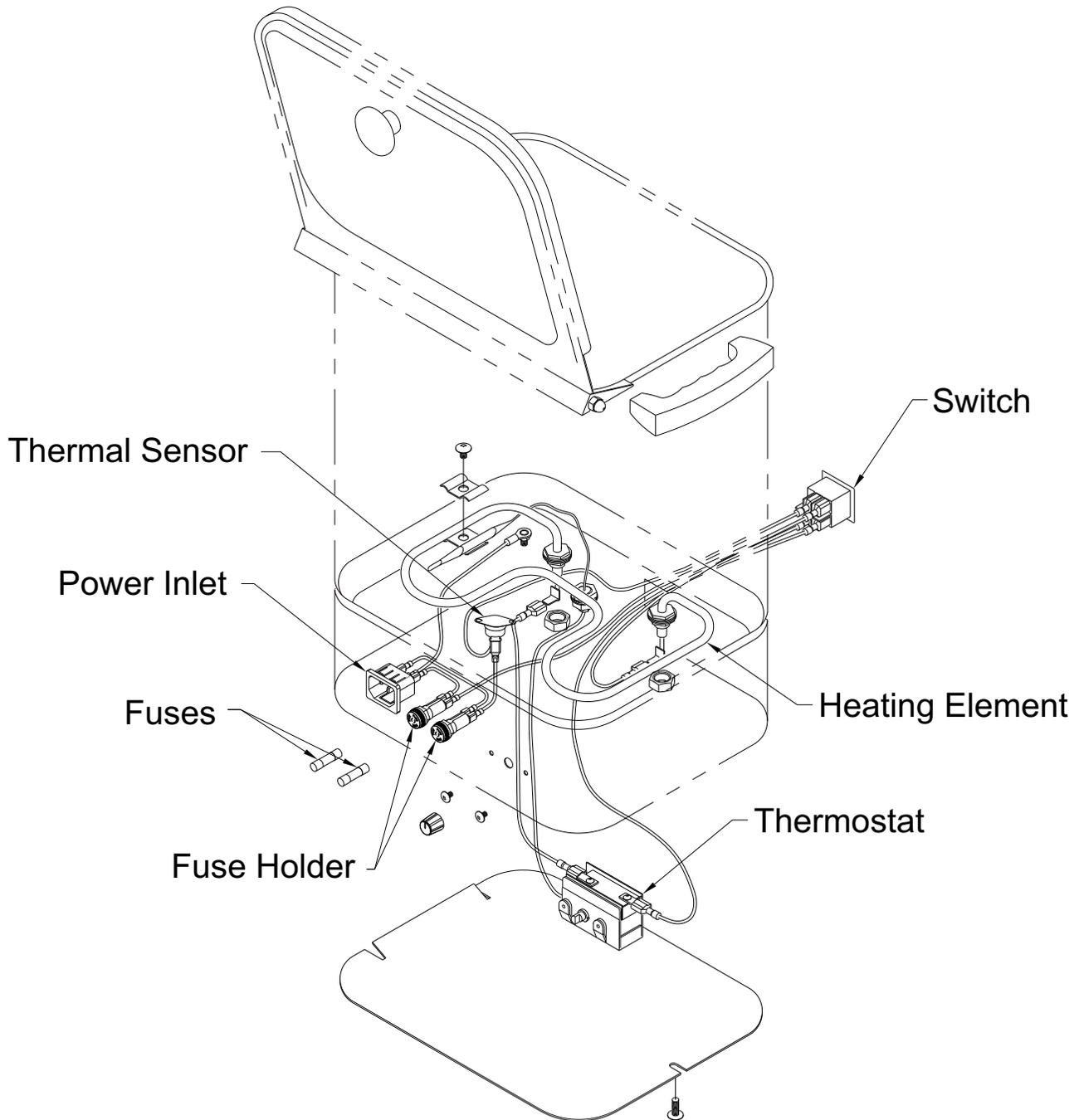



Whitehall Manufacturing®
Manufacturer of Healthcare and Rehabilitation Products since 1946

Mailing Address:

P.O. Box 3527 • City of Industry, CA 91744-0527 U.S.A

Physical Address:

15125 Proctor Avenue • City of Industry, CA 91746 U.S.A

Phone 800-782-7706 • 626-968-6681

Fax 626-855-4862

Web: www.whitehallmfg.com

E-mail: info@whitehallmfg.com

Any questions regarding servicing of products in Europe or the Middle East should be directed to:

Niva Medical Company • Olarinluoma 14 B, FIN-02200 Espoo, Finland

Phone 358 9 615 00 540 • Fax 358 9 615 00 545

Web www.nivamedical.fi • E-mail info@nivamedical.fi

EC REP



MANUFACTURER'S WARRANTY

Whitehall Manufacturing Company warrants that its products are free from defects in material or workmanship under normal use and service for a period of one year from date of shipment. Whitehall's liability under this warranty shall be discharged solely by replacement or repair of defective material, provided Whitehall is notified in writing within one year from date of shipment, F.O.B. Industry, California.

This warranty does not cover installation or labor charges, and does not apply to materials which have been damaged by other causes such as mishandling or improper care or abnormal use. The repair or replacement of the defective materials shall constitute the sole remedy of the Buyer and the sole remedy of Whitehall under this warranty. Whitehall shall not be liable under any circumstances for incidental, consequential or direct charges caused by defects in the materials, or any delay in the repair or replacement thereof. This warranty is in lieu of all other warranties expressed or implied. Product maintenance instructions are issued with each fixture, and disregard or non-compliance with these instructions will constitute an abnormal use condition and void the warranty. Stainless steel must be properly maintained after the water has been introduced into the fixture, or Whitehall's limited warranty is void. If you have any questions or require technical assistance, please call 800-743-8219.

NOTICE TO KEEP ORIGINAL PACKAGING- Regarding warranty claims: customer must retain original packaging for one year upon receipt of product. If packaging is discarded, it is the customer's responsibility to provide adequate packaging. Any shipping claims that are a direct result of customer-provided packaging materials will be handled by the shipper.